INSTRUCTIONS FOR USE

EN

Scarban Silicone sheets – Elastic and Light

Product Description

Scarban silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after skin surgery. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets glove and finger), washable and can be used for a long period of time.

Scarban® (Cleaning) Spots® is a cleaning agent for silicone sheets.

Composition

- Textile layer: polyamide, elastane
- Medical grade silicone

Properties

- Scarban silicone sheets ensure that scars become flatter, smoother and thinner and reduce complications (like persistent redness, pulling and itching, restriction of joint movement, discomfort and cosmetic concerns).
- The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which leads to an increased fluidity of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and that functional objectives (complications) are reduced.
- Scarban silicone sheets provide UV protection as standard (at least 93% UV blocked).

Indications/applications

Scarban silicone sheets are suitable for:

- all ages and all new scars
- keloid scars
- treatment and prevention of abnormal scars (hypertrophic and keloid)
- treatment of surgical scars
- use under clothing and in compression therapy
- treatment of persistent redness (erythema) caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns)

Instructions for use – Scarban silicone sheets

(For instructions on how to use the glove and finger, see below)

- For maximum effectiveness, Scarban silicone sheets should be applied immediately after wound closure (after removal of sutures and the disappearance of any scars).
- It is recommended that the Scarban silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.
- Scarban silicone sheets should be worn for 23 hours per day. Scarban sheets are used for a minimum of 12 and a maximum of 23 hours per day.
- If necessary, Scarban sheets can be cut to the correct size (fig. 1).
- Ensure that the Scarban sheets extend two centimetres beyond the edges of the scar.
- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar.
- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the areas surrounding it (fig. 2).
- Ensure that the silicone sheet fits from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).
- If necessary, the silicone sheet can be fixed in place with an adhesive tape (fig. 5).
- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Instructions for use – Scarban silicone sheet: Glove

- The Scarban silicone sheet glove is used for scarring on the hand or fingers.
- To put on the glove, first fold the cuff outward. This prevents the textile cuff from tearing from the glove when pulling it on and helps to prevent the glove from slipping off.
- If necessary, the finger (and especially the fingers) can be rubbed with a neutral, water-based lotion (cream) in order to allow the glove to slide easily over the fingers.
- Pull the finger sections of the glove over the fingers, using twisting movements.
- After donning, turn back the cuff of the glove.

Instructions for use – Scarban silicone sheet: Finger

(For instructions on how to use the glove and finger, see below)

- First determine the length of the silicone sheet as it needed, to fit the finger on the sheet. There are several colored markings on the side of the silicone sheet. The middle of the sheet should be cut at the bottom edge up to the colored mark at the end of the finger.
- First, the sheet with the silicone layer should be put on the finger.
- Afterwards, the finger can be rubbed with a neutral, water-based lotion (cream) to help it slide easily into the sheet.

Scarban Silicone sheets – Elastic and Light

- Pull the sheet over the finger with twisting movements.
- Then pull the sheet, consisting only of a textile layer, over the first silicone sheet. This is why there is an extra protective layer on top of the silicone sheet, which ensures optimal functioning of the silicone sheet.

Scar instructions for Scarban silicone sheets

(For maintenance of glove and finger sheets, see below)

- It is essential to clean the Scarban silicone sheets properly, on a daily basis. This is necessary to ensure good adhesion.
- Use the silicone sheets with a few drops of Scarban Silicone Cleaning Spots or a neutral soap (fig. 6).
- Remove as much dirt as possible with soap and lukewarm water.
- Rinse the silicone sheets thoroughly with lukewarm running water (fig. 7).
- After contact with soap and water, the silicone sheet will no longer be adhesive and it will pull up (fig. 8).
- The silicone sheet should be torn at the clean, dry surface, with the sticky side facing upwards (fig. 8).
- While drying, the silicone sheet will unroll and its adhesive strength will gradually return.
- Use the sheets only after it has completely dried.
- Trim edges of silicone sheets that are no longer adhesive, due to wear and tear.

Care instructions for Scarban silicone sheets: glove and finger

- In the washing machine: the Scarban silicone sheet glove and finger can be washed in the washing machine together with other clothing at 40 °C.
- Hand wash: the Scarban silicone sheet glove and finger can also be washed under the tap. Rub the inside of the silicone sheet with a few drops of Scarban Silicone Cleaning Spots or a neutral soap.
- Remove the dirt as much as possible with soap and lukewarm water.
- Rinse the silicone sheets thoroughly with lukewarm running water.
- Drying: dry the sheets on a clean, dry surface, away from direct sunlight or dry tea towel. Never dry the silicone sheets near or using a heat source.
- Use the silicone sheet only after it has completely dried.

How long should I use Scarban silicone sheets?

- It takes at least 3-6 months for a scar to normalise.
- Use the Scarban silicone sheet as long as the scar is raised, red or itchy.
- Consult your physician about the duration of the therapy.

Each pack of Scarban

- Each pack of Scarban contains one or more silicone sheets.
- If used, correctly and cared for properly (according to the instructions for use), each pack of Scarban is sufficient for approximately 4 to 8 weeks of treatment.
- In the event of long-term therapy, several packs may therefore be necessary.

Special attention and precautions

- Keep out of reach of children.
- Do not use Scarban silicone sheets on unsuitable for external use.
- Use Scarban silicone sheets only on healed, closed skin if the scar has disappeared and never on open wounds.
- Never dry the Scarban sheets in the sun or near a heat source.
- Temporarily remove the Scarban sheets if the patient is exercising or swimming. Perspiration or contact with water may cause the Scarban silicone sheet to detach.
- Do not use if you are hypersensitive to any of the ingredients.
- Do not use if the packaging is damaged when you receive the product.

Side effects

- In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated.
- Stop the treatment again and let the skin slowly get used to the Scarban silicone sheet (applying it for an additional hour every day).
- In skin problems persist, consult your physician.
- If there are any adverse effects related to the product, these should be reported to the manufacturer and to the competent authorities (if the Member State where the user is located).

Storage

- The expiry date is stated on the packaging.
- Scarban silicone sheets can be stored for 3 years in its unopened packaging.
- Store at temperatures between 5 and 25°C.

Date of latest revision of instructions for use

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