Scarban

Professional Scar Treatment

VELVET TOUCH

SILICONE SCAR GEL

Instructions for use

Scarban

Professional Scar Treatment

Because every scar needs care!

www.scarban.eu

EN

Title:
Scarban Velvet Touch silicone gel.

Product description:
Scarban Velvet Touch is a silicone gel for treating scars (hypertrophic and keloid) and for preventing abnormal scar development after surgery or an accident.

Composition:
Dimethicone, Dimethicone Crosspolymer, Cyclopentasiloxane, Cyclohexasiloxane.

Properties:

- Scarban Velvet Touch silicone gel makes scars flatter, smoother and thinner and reduces complaints (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns).
- Scarban Velvet Touch silicone gel is a transparent gel which, thanks to its unique formulation, dries quickly and does not feel sticky.
- The gel provides a semi-occlusive layer on top of the skin, creating increased hydration of the skin (epidermis). The increased hydration ensures that the damaged scar skin recovers faster and reduces complaints.

Indications/areas of application:

- Scarban Velvet Touch can be applied to both old and new scars for:
- the treatment and prevention of abnormal scars (hypertrophic and keloid).
 - the treatment of burn scars.
 - the treatment of surgical scars.
 - the treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic objections).

Before and after use

- Before use, twist the cap off the tube or remove the cap from the vial.
- If a tube, also cut off the top part using scissors, before use.
- After use, replace the cap on the tube or vial.

Instructions for use:

- For optimum results preferably start Scarban Velvet Touch silicone gel treatment immediately after wound closure (after removing sutures and the disappearance of any scabs).
- Clean the skin before use and allow it to dry.
- A small amount of silicone gel per treatment is sufficient. For optimal therapy, apply a small amount of silicone gel to the scar twice a day with a smooth motion.
- Reapply Scarban Velvet Touch silicone gel after intensive activity (sports) and contact with water (showering, swimming).
- If desired, Scarban Velvet Touch silicone gel can be combined with make-up after drying.

How long should I use Scarban Velvet Touch silicone gel?

- It takes at least 3-6 months for a scar to normalize.
- Use Scarban Velvet Touch silicone gel as long as the scar is elevated, red or causes complaints.
- Consult with the practitioner/doctor about the duration of the therapy.

Duration of use:

- When used correctly, the contents of a tube or vial are sufficient for several weeks of treatment.
- The duration of use depends on the size of the scar to be treated.
- Multiple packs may therefore be necessary in the case of long-term therapy.

Special attention and precautions:

- Keep out of the reach of children.
- Avoid contact with eyes and mucous membranes.
- The silicone gel is only suitable for external use.
- Use the silicone gel only on closed skin (after the scab has disappeared) and never on open wounds.
- Do not use in case of hypersensitivity to any of the ingredients.
- Do not use if the product is received with damaged packaging.

Side effects:

- In rare cases, the skin may react sensitively (skin irritation) to a silicone treatment. If this is the case, stop the treatment temporarily.
- If necessary, Scarban Velvet Touch can be removed using a mild, skin-friendly soap.
- Start the treatment again once the irritation has disappeared.
- If skin problems persist consult your practitioner/doctor.
- If there are any serious incidents related to the product these should be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

Storage:

- The expiry date is printed on the packaging.

Date of the last revision of instructions for use:
January 2024

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